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The image contains two instructional panels. The left panel, titled 'プライミング' (Priming), shows a syringe with a red dashed line indicating the fill level. Below it, a syringe is shown with a needle inserted, and red arrows point to the left, indicating the direction of flow. The text below reads: '先端まで栄養剤等を満たさないようにしてください。' (Please do not fill with nutrient etc. up to the tip). The right panel, titled '投与後' (After Administration), shows two syringes. The left one is crossed out with a red 'X' and has a blue circle around its tip. The right one is shown with a blue circle around its tip. The text above the syringes reads: '投与器具側を下側に' (Place the administration device side on the bottom). Below the syringes are two diagrams showing the syringe tip in a holder. The left diagram shows the tip pointing up, and the right diagram shows the tip pointing down. The text below the diagrams reads: 'コネクタを取り外す際は、投与器具側を下側にすることで、栄養剤等の垂れ込みを防止できます。' (When removing the connector, by placing the administration device side on the bottom, you can prevent dripping of nutrients etc.).

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